

FreiStil Tischlerei

Böhm und Kinstler GmbH & Co. KG

Ruppenkampstraße 16

D-49084 Osnabrück

Fon ++49 - 541-800 3969 0

Fax ++49 - 541-800 3969 99

E-mail info@freistil.com

Homepage: www.freistil.com



Operating Manual

Travel Bed "GULLIVER"



Read carefully before assembling!

1. ASSEMBLY INSTRUCTIONS

- 1.1 Join the long lateral pieces together as shown in the diagrams (Fig. 1 and Fig. 2) and screw them together with the screws (8 x 30mm) provided.



Fig. 1



Fig. 2

Tighten up all the screws and, after you have used the travel bed for some time, check to make sure they are still tight!

- 1.2 Join up the bottom, short lateral pieces with the upright posts as shown in the diagrams (Fig. 3 and Fig. 4) and screw them together with the screws (8 x 60mm) provided.

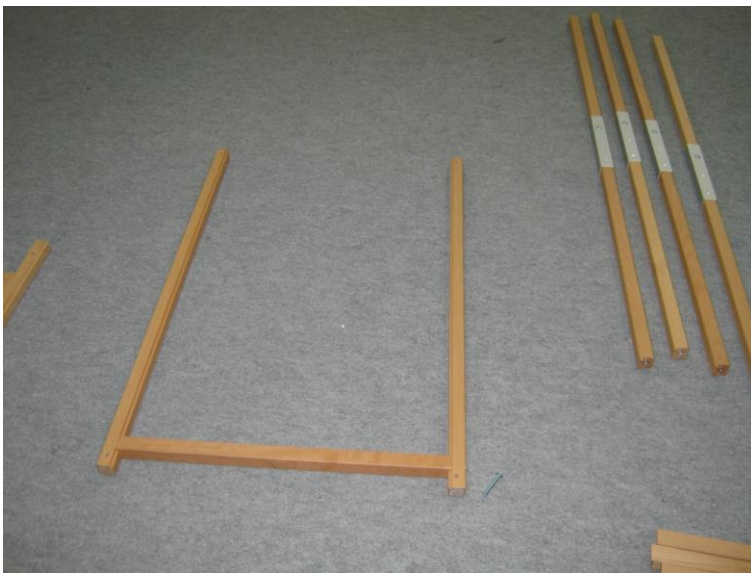


Fig. 3



Fig. 4

- 1.3 Connect the bottom, long lateral pieces (see 1.1) with the pre-assembled side pieces (see 1.2) as shown in the diagram (Fig. 5). Mount the rails designed to prevent the bed from falling over as shown in the diagrams (Fig. 5 and Fig. 6). If the travel bed is to stand with its long side against a wall, then the rails must be adjusted (offset) accordingly as shown (Fig. 7).



Fig. 5



Fig. 6



Fig. 7

- 1.4 Insert the top, long lateral pieces into the sleeves of the net as shown (Fig. 8) and connect the lateral pieces with the pre-assembled side pieces (see 1.2) as shown in the diagram (Fig. 9).



Fig. 8



Fig. 9

- 1.5 Insert the top, short lateral pieces into the sleeves of the net as shown (Fig. 10) and connect the lateral pieces with the pre-assembled side pieces (see 1.2) as shown in the diagram (Fig. 10).



Fig. 10

- 1.6 Fasten the net to the bottom, long lateral pieces via the Velcro tapes attached to the net.



Fig. 11

Manufacturer: **FreiStil Tischlerei**, Ruppenkampstraße 16, D - 49084 Osnabrück
Fon ++49 541-800 3969 0, Fax ++49 541-800 3969 99
email: info@freistil.com